



Tipping Hazard

RAMPART IC
M1128



DO NOT move or use Rampart without proper training

Case Sequencing

1. Unplug Rampart
2. Unlock Leg 2, Move to 90 Degree Position, Lock Back
3. Unlock Lower Rotation Lock (2 Clicks)
4. Unlock Upper Rotation Lock 1 (1 Click) While Opening the Extremity Panel to 180 Degrees (Straight Across)
5. Raise Extremity Panel Using Lower Row Buttons
6. Drape
7. Leg 3 Goes Under the Table
8. Do Not Unlock or Move Legs 1, 3, 4



Tips

- Leg Locks in Up Position = Locked
Down Position = Unlocked
- Extremity Panel to Sit ~2" from Patient Body
- RAK Flaps Pointing Towards Patient Feet
- Keep Patient R Arm as Close to Body as Possible
- Place Abdominal Shields on Under Table Lead
- Ensure Sheaths are Secure
- Move Mats and Cords Away from Rampart Rolling Pathways

End of Case

1. Lower Both (Radial and Extremity) Panels to Lowest Position
2. Fold Panels Together Over Legs 1 and 4 (Upper Lock Will Automatically Engage)
3. Move Leg 2 to Transit Position and Lock
4. Lock Lower Rotation Lock (Push All the Way In)
5. Lock Both (Upper and Lower) Rotation Locks
6. Plug in Overnight

